### 1 Open the device.

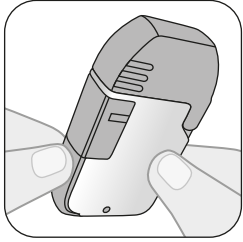
Hold the white base with one hand and pull right orange part handle to right side (capsule chamber) as shown. Be sure to open the right side (capsule chamber) completely until capsule filling chamber hole shown.

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### Insert the Rotacap into the capsule chamber.

Insert the Rotacap into the Rotaflo (capsule chamber)

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### 3 Close the side capsule chamber and press the side button.

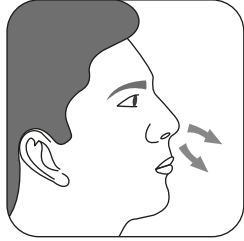
Close the side part (capsule chamber) and press to use.

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### Breathe out fully.

Breathe out fully before taking dose of Rotacap

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
### 5 Open the cap & breath in through mouth piece.

Grip the mouth piece between your teeth and seal your lips around it. Tilt your head slightly backwards. Breathe in through your mouth as deeply as you can. Remove the Rotaflo from your mouth and hold your breath as long as possible then breathe out.

Note: If you are doing this properly you will hear the Rotacap shell vibrating inside the Rotaflo.

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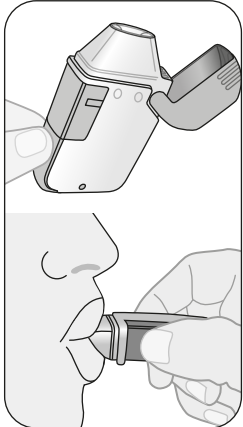
### 6 Open the capsule chamber.

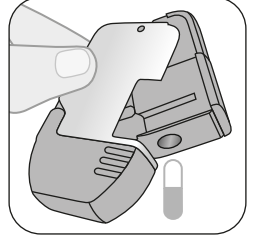
After every use, open the side part (capsule chamber) of Rotaflo and discard the used Rotacap.

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### 7 Close the cap & capsule chamber firmly.

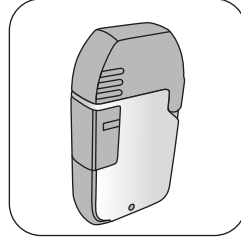Close the cap & capsule chamber. Store Rotaflo in convenient place. Every time use fresh Rotacap. Proceed with steps 1 through 6.

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
### The Rotaflo is robust and requires minimal cleaning and maintenance

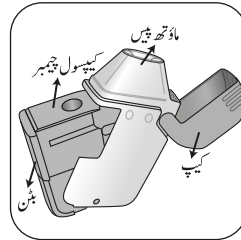- Wash the Rotaflo once every 2 weeks.
- Open the mouth piece.
- Remove any empty Rotacap shell that might be left in the chamber.
- Rinse the mouthpiece and chamber in running water.
- Remove excess water by tipping onto a paper towel and leave to air dry.
- Air drying takes about 6 hours so if possible wash the Rotaflo and leave it to dry overnight.

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### General Care Tips for your Rotaflo

- Do not push any cleaning cloth or instrument into the mouth piece of the Rotaflo as it may cause damage.
- Only insert the Rotacap into the device just prior to use.
- At times, a fine layer of powder may remain in the Rotacap shell after use. This does not affect the efficacy of the medication.
- Use only as directed by your doctor. If the recommended dose does not provide relief or symptoms become worse, consult your doctor.
- It is recommended to replace your Rotaflo every 6 months.

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